**SCR2795 – ASTA PORTAFLEBO 4 GANCI – I.V. STAND 4 HOOKS**  
**SCR2795/P – ASTA PORTAFLEBO 2 GANCI – I.V. STAND 2 HOOKS**  
**SCR2795/A – ASTA PORTAFLEBO ALLUMINIO 2 GANCI – I.V. STAND**  
**ALLUMINIUM 2 HOOKS**

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**ATTENTION!** The operators must carefully read and completely understand the present manual before using the product.

**DESCRIPTION**

Telescopic I.V. stand with 2/4 hooks for the support of bags and infusions.  
 Composed of two coaxial tubes in chromed or aluminum steel. Tubular adjustable up to 210 mm by plastic regulation crank. Equipped with black plastic base with 5 wheels Ø 50 mm.  
 The device is an aid for the care of the patient who is bedded or with limits functional autonomy.

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**SCR2795**

## ASSEMBLY INSTRUCTIONS

1. Take the wheels and insert them into the base.
2. Take the tubular with a larger section (25x1.2) equipped with a knob, insert it first into the black cap, then into the base, fix the parts, and screw it into the area below.
3. Take the tubular section with 20x1 and insert the tube inside the other until it stops using the plastic regulation crank.
4. For SCR2795/P-SCR2795/A codes, take the plastic support with two hooks and press to insert the cap at the end of the tubular. If necessary, use a soft hammer to insert it all completely.
5. For SCR2795 code, fix the support with four hooks by knob supplied.

## TECHNICAL DATA

	<b>SCR2795</b>	<b>SCR2795/P</b>	<b>SCR2795/A</b>
<b>Measurement</b>	Adjustable H 160-210 cm	Adjustable H 160-210 cm	Adjustable H 165-210 cm
<b>Weight</b>	3,150 kg	2,900 kg	2,080 kg
<b>Max capacity</b>	Steel hook, <b>2 kg</b> / each	Steel hook, <b>5 kg</b> / each	Plastic hook, <b>5 kg</b> / each

## WARNINGS!

- **Fix well with the supplied screws.**
- **Check the locking nuts are tightened.**
- **Make sure that the parts are locked in their housing.**
- **Observe the maximum flow rates indicated in the data sheets.**
- **Avoid direct contact with open wounds.**
- **Not for use in open environments.**

## MAINTENANCE AND CLEANING

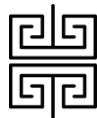
- Use only original spare parts.
- Periodically check all fixing points.
- To preserve at best, it is recommended to avoid prolonged exposure to sunlight, to contact and deposit in saline and very humid environments.
- Wash the stainless-steel surfaces with soapy water and use appropriate disinfectants.
- Rinse with a damp cloth and dry thoroughly.
- Do not spill on the surface cleaners containing chlorine or ammonia, fatty and oily substances.
- Do not use cleaning utensils containing iron. If necessary disinfection only use suitable non-corrosive products.

## WARRANTY

- The product is guaranteed for ONE YEAR for defects in materials and workmanship, excluding consumables.
- Tampering or damage caused by improper use of the product implies the immediate loss of guarantee.
- This medical device was created exclusively for the above-mentioned methods. Any other use is to be considered improper.
- We cannot be held liable for damages to persons or things due to improper use of the garrison.

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Please contact the manufacturer for information and technical assistance:

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Keep in a cool, dry place

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Keep away from sunlight

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Please read instructions carefully

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Product complies with Directive (UE) 2017/745 (MDR)

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