SCR2803 – LETTO A VALIGIA

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ATTENTION! The operators must carefully read and completely understand the present manual before using the product.

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This item is a folding couch in chromed steel with a suitcase, resin padding and fire-retardant fabric covering (black colour). Supplied with a velvet bag for easy transport. Designed for home visits or therapies in clinics where are no couch.

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1. Lay down the suitcase with the handle on the upper side.
2. Open the suitcase.
3. Open the legs of the bed completely.
4. Only when the legs are completely open and fixed, Turn the bed in the correct position of use.

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- **Not for use in open environments.**
- **When adjusting the height of the couch and the back, make sure that no one inserts hands, legs or anything into the movement area: CRUSHING RISK!**
- **Lay the patient only when the net (seat and back) is completely horizontal.**
- Move the bed without lifting it from the floor.
- Before moving the bed, unlock the wheels.

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- Use only original spare parts.
- Periodically check all fixing points.
- To preserve at best, it is recommended to avoid prolonged exposure to sunlight, to contact and deposit in saline and very humid environments.
- Wash the stainless steel surfaces with soapy water and use appropriate disinfectants.
- Rinse with a damp cloth and dry thoroughly.
- Do not spill on the surface cleaners containing chlorine or ammonia, fatty and oily substances.
- Do not use cleaning utensils containing iron. If necessary disinfection only use suitable non-corrosive products.

Cleaning rules for coating:

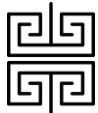
- Clean with damp soapy cloth and rinse well with clean water. Do not use solvents, bleaches, synthetic detergents, polishes or aerosol spray. Attention: Normally synthetic leather light colours should not come in contact with no fixed coloured clothes (such as jeans and similar) not to cause stains and rings that cannot be cleaned up.

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- Tampering or damage caused by improper use of the product implies the immediate loss of guarantee.
- This medical device was created exclusively for the above-mentioned methods. Any other use is to be considered improper.
- We can not be held liable for damages to persons or things due to improper use of the garrison.

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Keep in a cool, dry place

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Keep away from sunlight

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Please read instructions carefully

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Product complies with Directive (UE) 2017/745 (MDR)

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